

# Immediate Denture Patient

## Overview

Taking good records is key when making an immediate denture for a patient who has teeth planned for extraction. Following these steps carefully helps doctors make a denture that looks good and works well for the patient right after their teeth are taken out, making the process easier and helping them heal.

### TECHNOLOGY & MATERIALS

- ☐ IOS scanner or PVS impression material
- ☐ *Optional:* Blue or green mousse - for bites only

## Immediate Denture Workflow

1. Take the following photos with the patient standing up:

- Full-face full-smile
- Full-face exaggerated smile
- *Optional:* Full-face profile



2. Capture upper and lower impressions and bite. You may either:

- Capture IOS scan of upper, lower, and bite, OR
- Capture physical impression and bite



3. Send your records to ROE and complete the Rx via [www.roedentallab.com/upload](http://www.roedentallab.com/upload). If using IOS, you may submit your scans via the IOS portal, OR export your .STLs and upload to the ROE portal.



4. Seat the new denture, post-extractions.



5. If the denture does not seat, you may either:

- Do a post-healing reline, OR
- Make a new denture following the “Patient with Reference Denture” protocol (see page 58).



## Your Dedicated Support

When you partner with ROE, you are given a dedicated, experienced Technical Support and Case Support Specialist team who is personally responsible for your account. These one-on-one relationships ensures that ROE follows your explicit instructions and your quality standards.

To learn more, visit [www.roedentallab.com/TSS](http://www.roedentallab.com/TSS)



URL:  
Dedicated  
Support

Christine

